CR-Dosierpipette (kindersicher) / CR-Dosing pipette (child resistant)

EU-Patent

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*The patented "CR-Dosing Pipette" is the first and only absolutely child proof dosing pipette that combines all the advantages of the **elm-plastic** dosing pipettes, like easy handling, dosing accuracy and stability. The pipette is not accessible for children because before and after usage, it is kept inside the bottle with the drug, closed with a conventional PP28-CR-screw cap. When opening the screw cap, the CR-Dosing Pipette is automatically ready for use, thanks to the patented technology.*

Dosing pipette handling procedure

1. Undo and remove the standard CR sealing cap (push&turn) from the bottle (Fig. 1). The pipette is ready and child resistant in the bottle.
2. The springy plunger now appears automatically (Fig. 2).
3. At the start of the measuring procedure, the transparent dosing body as well as the white plunger are in the bottommost position. To measure the dosing quantity, use one hand to hold the bottom transparent dosing body and the other hand to pull on the top white plunger until you can read the desired quantity in ml or the body weight (Fig. 3).
4. Pull the pipette by the dosing body (bottom transparent ring) out of the bottle (Fig. 4).
5. Put the quantity removed from the dosing pipette onto a spoon by pressing down on the plunger (white ring) or empty into a glass filled with some liquid and give it to the patient. The liquid can also be given straight from the pipette into the patient's mouth.
6. Push the empty dosing pipette back into the bottle until the next application / usage.

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elm-plastic^{GmbH}