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
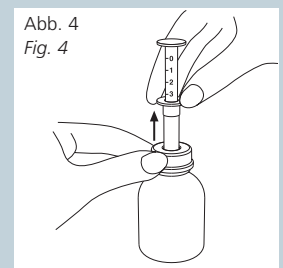
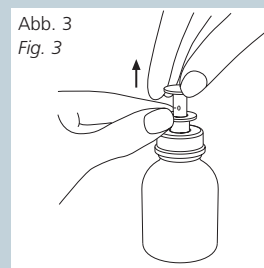
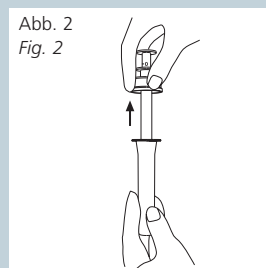
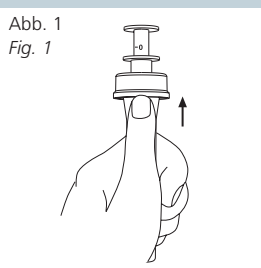
Holed screw cap and protective sheath Material: PE

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For bottle threads PP25, PP28 and PP31, compatible for all pipette sizes.

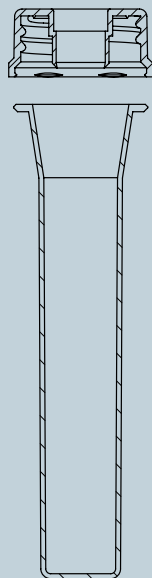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Handling procedure for dosing pipette with holed screw cap and protective sheath

1. Undo and remove the sealing cap of the bottle.
2. Using thumb pressure, detach the protective cap of the pipette from below (Fig. 1) and then pull out the pipette (Fig. 2).
3. Screw the pipette with the holed screw cap onto the bottle. The pipette is now ready for use and remains screwed on the bottle.
4. At the start of the measuring procedure, the transparent dosing body as well as the white plunger must be in the bottommost position. To measure the dosing quantity, use one hand to hold the bottom, transparent dosing body and the other hand to pull on the top, white plunger until you can read the desired quantity in ml or the body weight (Fig. 3).
5. Pull the pipette by the dosing body (bottom, transparent ring) out of the bottle (Fig. 4).
6. Put the quantity removed from the dosing pipette onto a spoon by pressing down on the plunger (white ring) or empty into a glass with some liquid and give it to the patient. The drug can also be given directly from the pipette into the patient's mouth.
7. Push the empty dosing pipette into the holed screw cap of the bottle until it is used the next time.